

PharmaJet®



Injector-0.5mL



Reset Station



Syringe-0.5mL
Needle-free



Filling/Vial
Adapter

Helpful Tips

Important:

Before you use your PharmaJet Stratis Needle-free Injection device, please read the Instructions For Use and these Helpful Tips completely. Follow the directions carefully when using the Stratis device.



Refer to Instructions for Use to ensure safe injections and to review risks.

Identify if the Injector is in the charged or uncharged state by confirming the position of the Injector Hammer.



Uncharged



Charged

Ensure the Injector is fully nested in the Reset Station, and that the pin is in line with the Injector as shown below.



DO NOT dry fire the Injector. Repeated dry firing will damage the Injector. Dry firing is activating the Injector with either an empty Syringe or without a Syringe. DO NOT attempt to use a partially filled Syringe; damage may occur.




1. Prepare the Injector

Place Injector nose first into the Reset Station.



Push Injector forward to fit into place.

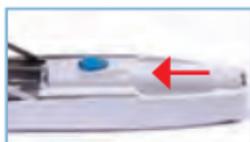

Make sure the pin is in line with Injector.



STAND and push **FIRMLY** on the lid of the Reset Station.



Continue to push down **firmlly** against the **resistance** until lid closes completely. Lid will **lift** indicating Injector is charged.



2. Fill Syringe



Remove blue protective cap from Syringe.



Place Syringe into top of Vial/Filling Adapter. Push down and twist to lock Syringe into place.



Insert spike downward into vial with a twisting motion.



Invert Adapter/Syringe assembly, pull down on Syringe plunger to draw fluid into the Syringe. Remove air bubbles as required.



Complete filling by pulling down on the plunger until **red O-ring** is below the ridge (fill line).



Snap off end of plunger at neck and discard.

3. Load Injector

Hold Injector as shown.

Remember to keep **thumb** on Multi-function Button.



While pressing down on the Multi-function Button, insert Syringe assembly, red O-ring first, into Injector until you hear a "click" locking it into place.

Keeping thumb on Multi-function Button, invert assembly and remove Syringe from Vial/Filling Adapter with a counterclockwise turn.



4. Give Injection

Remove Thumb off Multi-function Button

Patient assessment and preparing the Injection site location should be performed per CDC guidance.

Place Syringe flat against the injection site at a 90 degree angle. Apply gentle pressure until blue retractable core stops moving.

Pinch up skin over the injection site to limit depth of penetration.

Press down on the Multi-function Button to inject. A "click" will be heard upon injection.

Press down on the Multi-function Button to inject. A "click" will be heard upon injection.

Intramuscular Injection



Subcutaneous Injection



OR

Immediately following injection apply pressure to injection site for approximately 60 seconds.



Hold Multi-function Button and recap Syringe.

Push Button **inward** to eject Syringe. Discard used Syringe.



For Additional Support

E-mail: support@pharmajet.com or Call: 888-900-4321

Doc. #60-10449-001A